

Summit®  
...responsible solutions.

# tick•take

TICK REMOVER TOOL

REMOVE & TAKE AWAY TICKS SAFELY—  
without dropping or touching them!

## How to use:


1 Engage the slot of the **Tick•Take** by approaching the tick from the side (if possible), front or back until it is evenly aligned in the notch of the scoop.

2 Once tick is positioned in notch — press just slightly down on the skin and with even pressure *continuously slide the tool smoothly in the same direction* until the tick detaches.

▶ The specially designed **Tick•Take** notch grabs the tick at skin level and *with one continuous sliding motion* removes it.

3 Once tick is fully detached and contained in scoop carefully dispose by dropping in some rubbing alcohol or save for testing.

4 Clean the wound thoroughly with alcohol or similar disinfectant, or clean water and soap before and after each use.

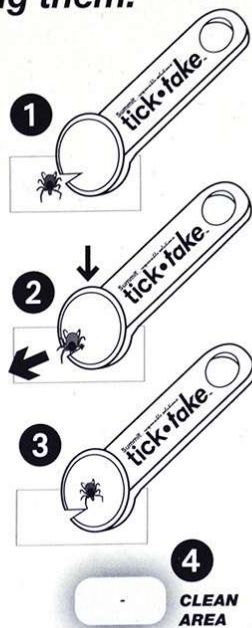

Remember to **clean the Tick•Take**  
with alcohol **after each use.**

Also use the **Tick•Take** spoon to quickly scoop up crawling ticks for disposal and reduce the risk of tick-borne diseases.



Summit  
...responsible solutions.®

235 South Kresson Street  
Baltimore, Md 21224

800-227-8664 | 410-522-0661

© Registered Trademarks of Summit Chemical Co.